
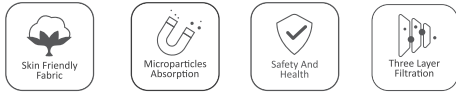


# Technical Data sheet

## MAXAM Masks, MX-220 Disposable MEDICAL Face Mask

A 3ply face mask is a loose-fitting, disposable device that creates a physical barrier between the mouth and nose of a 3ply face mask is a loose-fitting, disposable device that creates a physical barrier between the mouth and nose of the wearer and potential contaminants in the immediate environment.

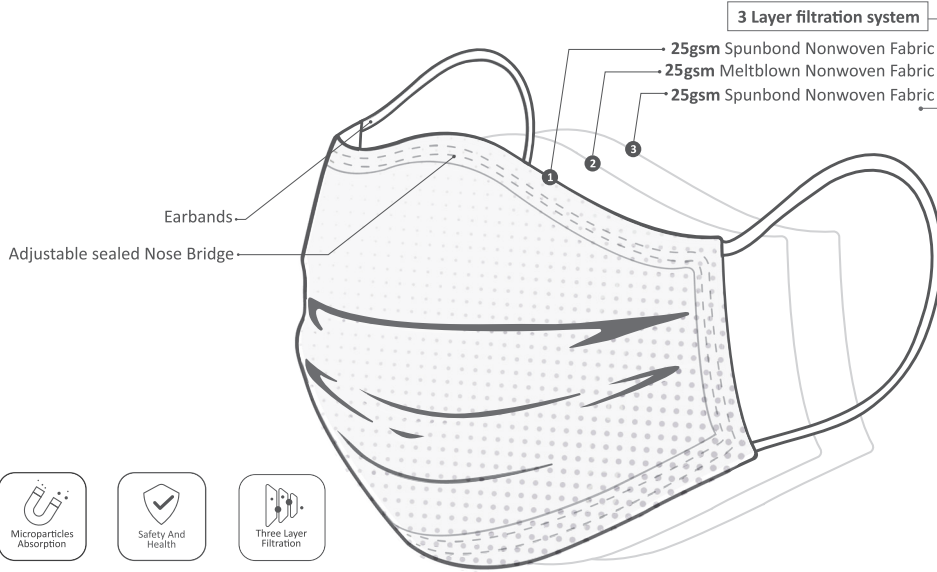

**PRODUCT**

**PRODUCT NAME:**

**CLASSIFICATION:**

**MODEL:**

**QUALITY ASSURANCE AND CONFORMITY:**

**MEDICAL FACE MASK**

**MX-MFM**

**Type IIR**

**MX-220 (Disposable 3-ply Medical Non Sterile Face Mask)**

**Directive 93/42/EEC**

Regulation: **(EU) 2017/745**  
 of the European Parliament and of the Council of 5 April 2017 on  
 Medical Devices  
**EN 14683:2019+AC:2019 Annex B**  
 Bacterial Filtration Efficiency (BFE)  
**EN 14683:2019+AC:2019 Annex C**  
 Brethability (Differential Pressure)  
**ISO 22609:2004**  
 Resistance against penetration by synthetic bloo  
**EN ISO 11737-1: 2018**  
 Microbial cleanliness (Bioburden)  
 Test report: **AR-20-YL-004978-02 (2020 08 26) EUROFIN**

**MATERIALS:**

Outer Material	<b>1st layer - 25gsm Spunbond Non-Wowen</b>
Filter Layer	<b>2nd layer - 25gsm Melt-Blown fabric</b>
Inner Material	<b>3rd layer - 25gsm Spunbond Non-Wowen</b>
Nose-piece	<b>Plastic single wire thread 3mm</b>
Earloops	<b>3mm Polyester and Spandex</b>

**SIZE:**

Mask body **175x95mm (+/- 5mm)**

**SHELF LIFE:**

**3 years after production date**

**STORAGE:**

**Storage under cool, clean, and dry conditions. Avoid excessive heat ( -20°C to +40°C and at less than 80% relative humidity )**

 Usage instruction



1. Before opening and touching the mask, wash your hands thoroughly with soap and water or alcohol/based hand sanitizer (>60% alcohol content).
2. Open the mask with both hands to the two sides.
3. Hold the mask by the ear loops. Place a loop around each ear and be sure the nose clip is upwarded
4. Press the nose clip to fit the shape of the nose
5. Pull the bottom of the mask over your mouth and chin

 Precautions

1. This product is not suitable for protection against harmful gases, steam and oily aerosols.
2. The product does not generate oxygen and is not suitable for use in anoxic environments, underwater operations, during evacuation or firefighting environments.
3. It is not recommended to use this protective mask if you do not feel a good adhesion between the face and the edge of the mask during use.
4. If found during use that the mask is damaged or the breathing resistance is too large, replace with a new one.
5. This product is for one-time use, masks should not be reused.